PHYWE Systeme GmbH & Co. KG
Robert-Bosch-Breite 10
D-37079 Göttingen

Telefon +49 (0) 551 604-0
Fax +49 (0) 551 604-107
E-mail info@phywe.de
Internet www.phywe.de

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Abb. 1: curricuLAB

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The screenshot shows the homepage of curriculab.de. At the top, there are navigation links for "Anmelden" (Login) and "Registrieren" (Register), and a language selection button with the German flag. The main heading is "curriculAB® Einfach. Digital. Unterrichten." Below it, a sub-headline reads "Die ausgezeichnete* Lehr- und Lernplattform für Naturwissenschaften". To the right, there is a circular seal for "Comenius EduMedia Medaille 2022". Further down, there are three mobile device screens displaying different parts of the platform: a list of experiments, a video of students in a lab, and a detailed experiment description for "Wechselspannungserzeugung" (Generating AC voltage). A note at the bottom states: "* curriculAB® wurde als pädagogisch, inhaltlich und gestalterisch herausragendes Lehr- und Lernmanagementsystem mit dem Comenius-EduMedia-Siegel 2020 ausgezeichnet."

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This screenshot shows the same homepage as above, but with a large red button at the bottom center labeled "Jetzt ausprobieren" (Try it now) in white text. The rest of the page content, including the experiment descriptions and the Comenius seal, is visible above the button.

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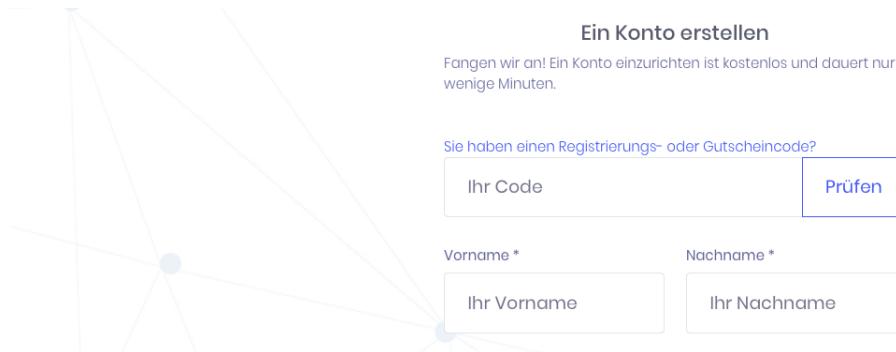
The screenshot shows the login page of the curricuLAB® website. The header includes the curricuLAB® logo and the PHYWE logo. The main content area has a blue background with a network-like pattern. It displays a welcome message:

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[Anmelden](#)

Below this, there is a larger section with the text:

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On the left, it says "Hallo Max, Willkommen bei curricuLAB!" and features a blue circular icon with a smiley face. On the right, there is an illustration of a lightbulb and various scientific symbols like "Sci", "Phy", and "Chem". At the bottom, there is a note:

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
The screenshot shows the top navigation bar of the curricuLAB website. It includes the curricuLAB logo, the PHYWE logo, and navigation links for 'Start', 'Inhalte' (which is highlighted in blue), 'Freigaben', and 'Statistiken'. A red 'neu' button is also visible. On the right side, there is a greeting 'Hallo, Max MM' and the PHYWE logo with the tagline 'excellence in science'.

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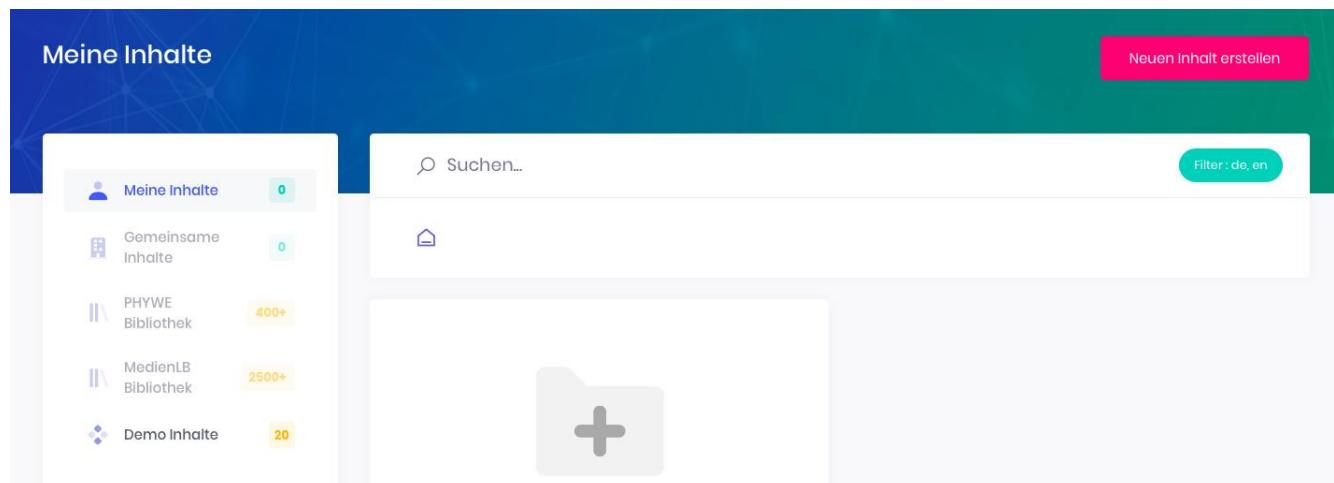
The screenshot shows the 'Meine Inhalte' section. On the left, there is a sidebar with icons and counts for 'Meine Inhalte' (0), 'Gemeinsame Inhalte' (0), 'PHYWE Bibliothek' (400+), 'MedienLB Bibliothek' (2500+), and 'Demo Inhalte' (20). The main area features a search bar with placeholder 'Suchen...' and a large 'Neuen Inhalt erstellen' button. Below the search bar is a folder icon with a plus sign, indicating where new content can be added.

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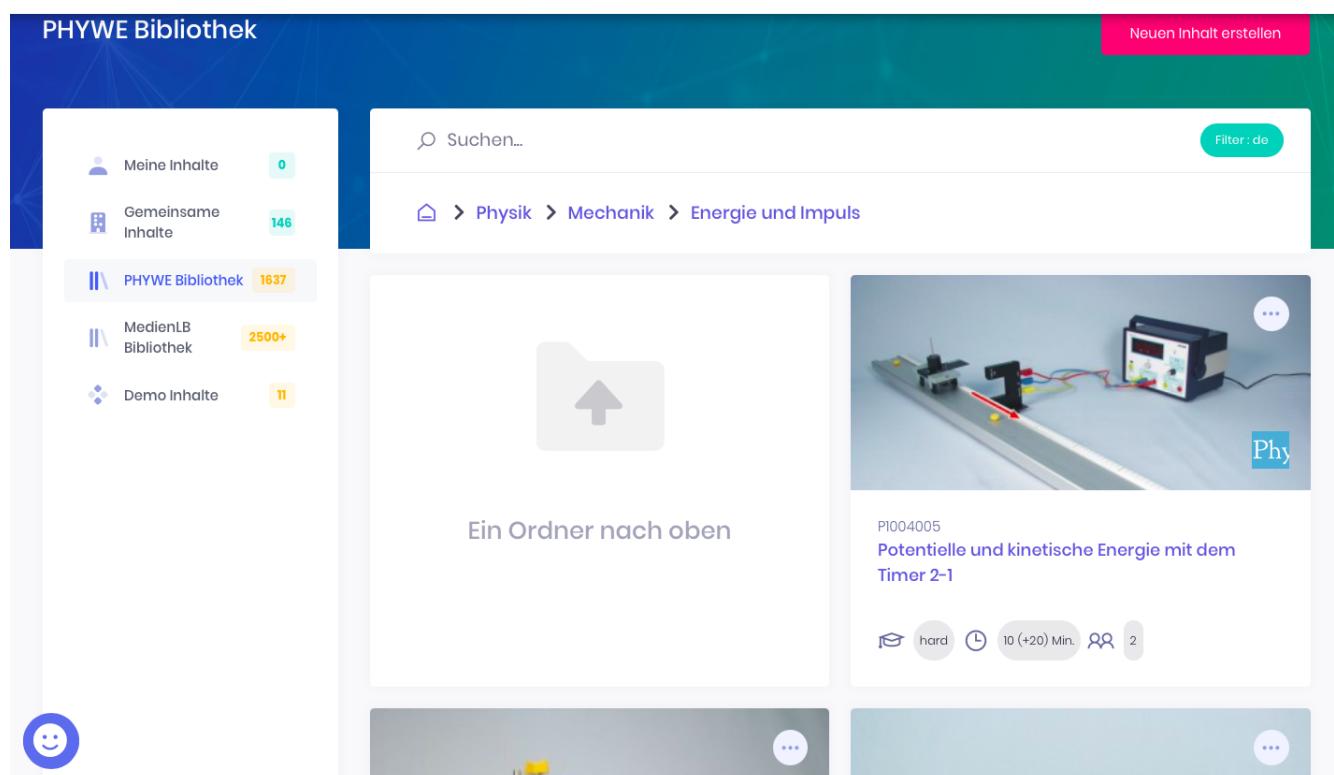The screenshot shows the search results for "kinetische Energie". On the left sidebar, there are links to "Meine Inhalte" (0), "Gemeinsame Inhalte" (146), "PHYWE Bibliothek" (1637), "MedienLB Bibliothek" (2500+), and "Demo Inhalte" (11). The main search bar contains the query "kinetische Energie". Below it, the results are listed with two items:

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Each result card includes a small image, the experiment number, a title, the category "Phy", and a detailed info button.

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The screenshot shows the context menu for experiment P1004005. The menu options are:

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
The screenshot shows the extended context menu for experiment P1004005 in the "Meine Inhalte" section. The menu options are:

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
A large "Neuen Ordner erstellen" button is visible on the right side of the screen.

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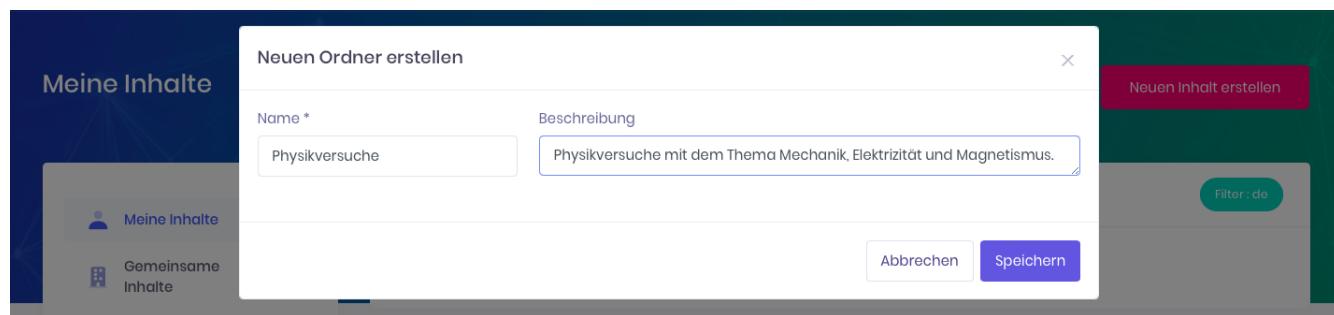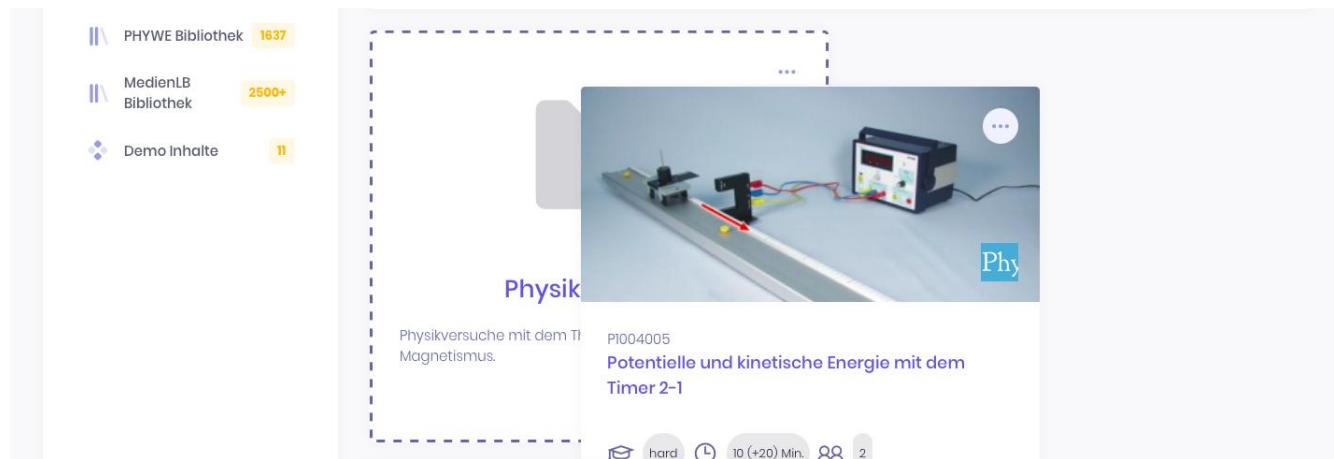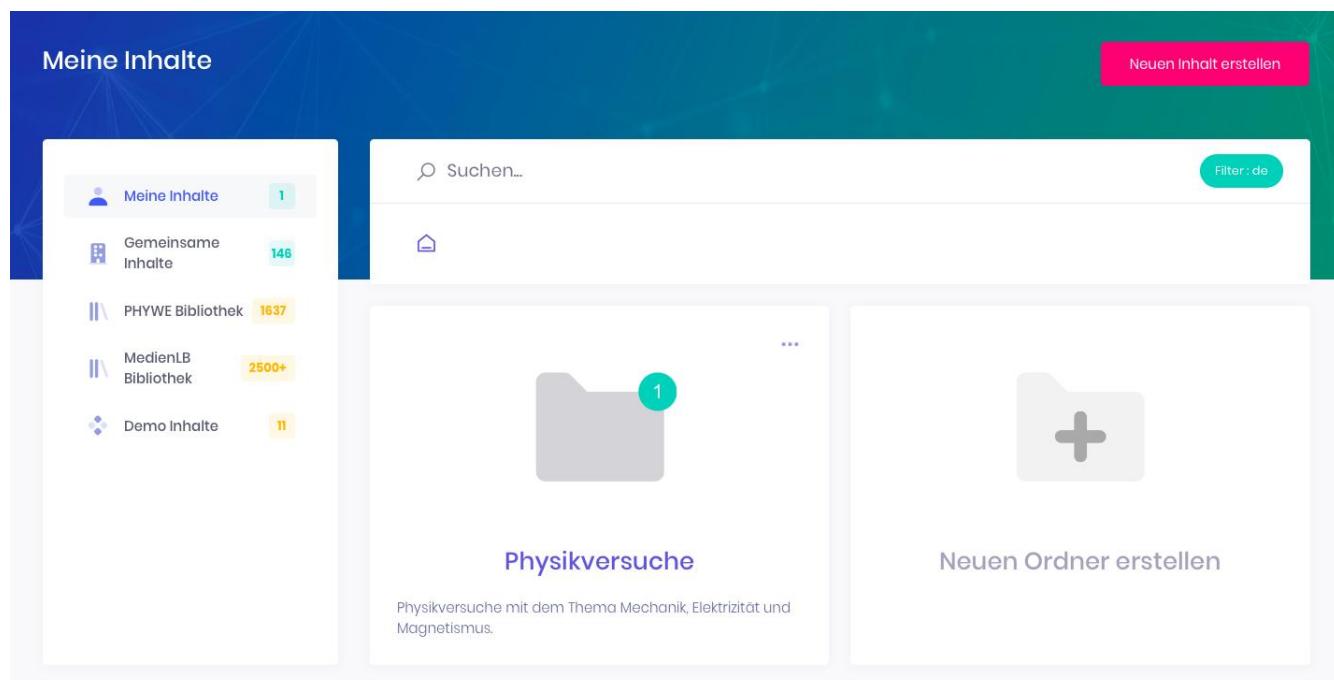
The screenshot shows the top navigation bar of the curricuLAB website. It includes the curricuLAB logo, the PHYWE logo, and navigation links for 'Start', 'Inhalte' (which is highlighted in blue), 'Freigaben', 'Statistiken', and a red 'neu' button. On the right side, there is a user profile for 'Hallo, Max MM' and the PHYWE logo with the tagline 'excellence in science'.

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The screenshot shows the 'Meine Inhalte' (My Content) page. On the left, there are four categories: 'Meine Inhalte' (1 item), 'Gemeinsame Inhalte' (146 items), 'PHYWE Bibliothek' (1637 items), and 'MedienLB' (several items). On the right, there is a search bar with placeholder text 'Suchen...', a filter button labeled 'Filter: de', and a large pink button labeled 'Neuen Inhalt erstellen'.

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The screenshot shows the 'Inhalt erstellen' (Create Content) editor. At the top, there is a toolbar with a 'Speichern' (Save) button. Below it, the 'Editor' section has a title 'H5P hub Inhaltstyp auswählen' with two radio buttons: 'Inhalt erstellen' (selected) and 'Hochladen'. There is also a 'Einfügen' (Insert) button. A search bar 'Nach Inhaltstypen suchen' is present. The main area displays a list of 'Alle Inhaltstypen' (All Content Types) with 48 results, sorted by 'Beliebteste zuerst' (Most Popular first). The list includes:

- Interactive Video**: Create videos enriched with interactions. Status: Aktualisierung verfügbar (Update available). **Details** button.
- Course Presentation**: Create a presentation with interactive slides. Status: Aktualisierung verfügbar (Update available). **Details** button.
- Multiple Choice**: Create flexible multiple choice questions. Status: Aktualisierung verfügbar (Update available). **Details** button.
- Quiz (Question Set)**: **Details** button.

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The screenshot shows the H2P hub Course Presentation editor interface. At the top, there is a header bar with the text "Editor" and the H2P hub logo. Below the header, the title "Course Presentation" is displayed. A toolbar below the header includes icons for "Tutorial" and "Beispiel". The main content area shows a list item titled "Das Mikroskop". Underneath the list item, there is a title input field labeled "Titel * Metadaten" with the text "Das Mikroskop" entered. A note below the input field states: "Wird für die Suche, Berichte und Urheberrechtsinformationen benutzt". At the bottom of the content area is a toolbar with various icons for editing and managing the slide.

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The screenshot shows the H2P hub Course Presentation editor interface in edit mode. The main content area is currently empty, represented by a large white space. Below the content area is a toolbar with various icons for editing and managing the slide, including text, image, video, and other media options. A small circular icon with a smiley face is visible in the bottom left corner of the slide area.

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

The screenshot shows the H2P hub Course Presentation editor interface. The toolbar at the top includes an icon for "Image" (a camera symbol) which is highlighted with a blue border. The main content area contains the title "Das Mikroskop" and a note about its use for search, reports, and copyright information. Below the content area is a toolbar with various icons for editing and managing the slide.

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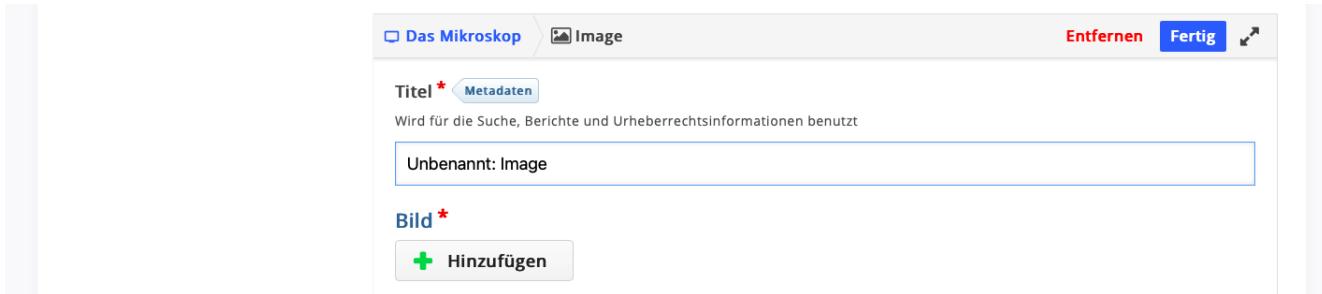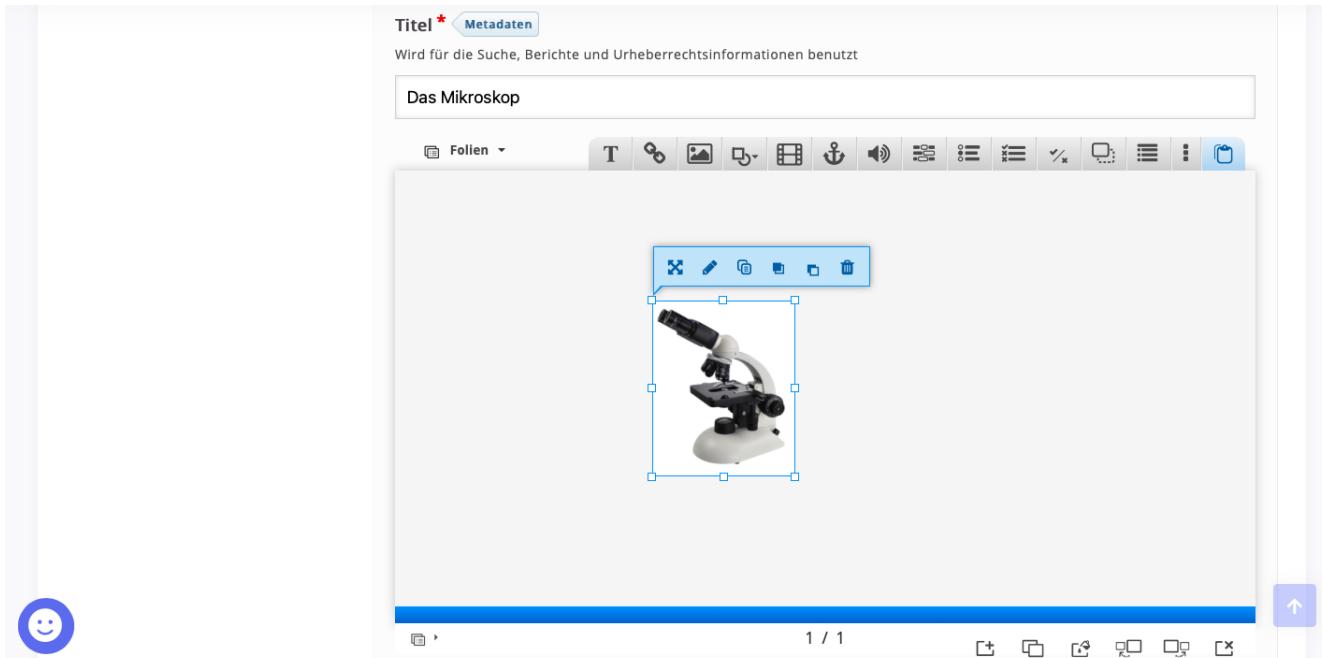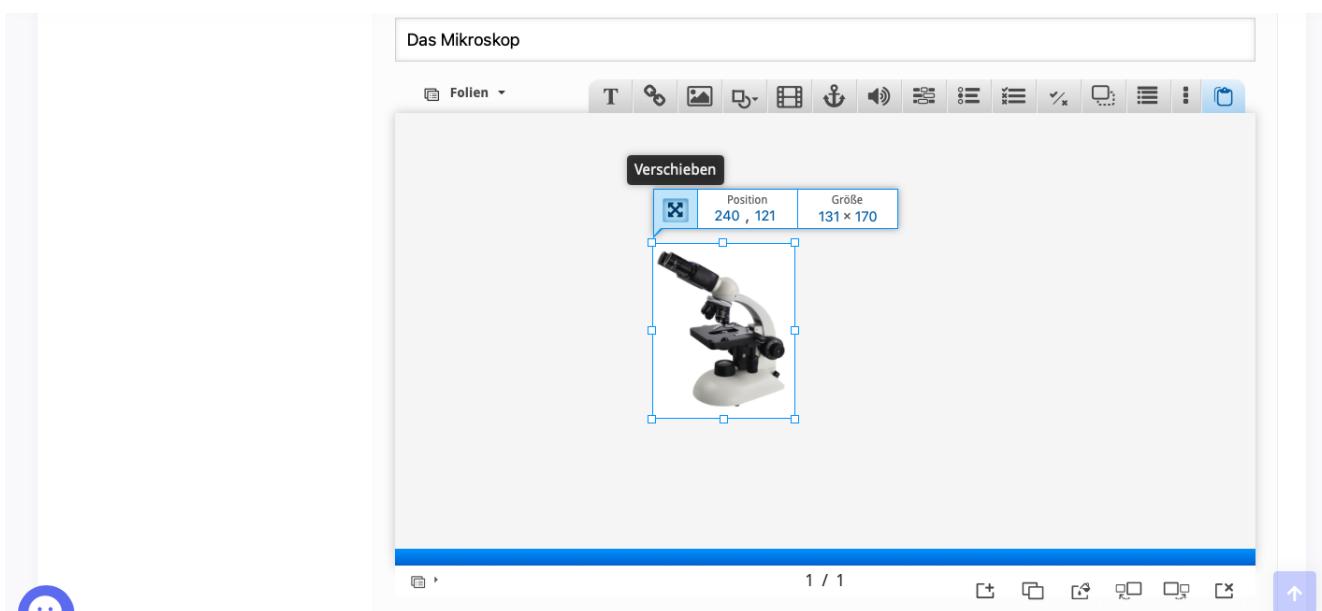
The screenshot shows a software window titled "Das Mikroskop" with a sub-tab "Unbenannt: Text". It includes a "Titel * Metadaten" section and a "Text *" section. The text area contains sample text about microscopes. At the top right, there are buttons for "Entfernen" and "Fertig".

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The screenshot shows a slide with a title "Das Mikroskop" and a text box containing sample text. The text box has a blue selection border with handles at the corners and sides. To the left of the slide content is a microscope image. At the bottom right of the slide are navigation icons for a smiley face, back, forward, and search.

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The screenshot shows the 'curricuLAB® PHYWE' interface for creating a new content item. At the top, there are navigation links: Start, Inhalte, Freigaben, Statistiken, and a 'neu' button. On the right, it says 'Hallo, Max MM PHYWE excellence in science'. Below the header, there's a 'Beschreibung' field with the placeholder 'Kurze Beschreibung des Inhalts'. Under 'Vorschaubild', there's a grid of icons representing various lab equipment like flasks, microscopes, and scales, with a 'Bild auswählen' button. A section for 'Themengebiete' includes checkboxes for Phy (blue), Che (orange), Bio (green), MINT (teal), and Sci (red). Under 'Speicherort', there are radio buttons for 'Meine Inhalte' (selected) and 'Gemeinsame Inhalte'. The 'Unterordner' section shows a folder path: 'Physikversuche'. At the bottom, there's a blue smiley face icon, a 'Speichern' button, an 'Abbrechen und zurück' button, and an upward arrow icon.

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The screenshot shows the 'Meine Inhalte' (My Content) page. At the top, there are navigation links: Start, Inhalte, Freigaben, Statistiken, and a 'neu' button. On the right, it says 'Hallo, Max MM PHYWE excellence in science'. A pink button on the right says 'Neuen Inhalt erstellen'. On the left, there's a sidebar with categories: 'Meine Inhalte' (2), 'Gemeinsame Inhalte' (145), 'PHYWE Bibliothek' (1644), 'MedienLB Bibliothek' (2500+), and 'Demo Inhalte' (11). The main area shows a search bar with 'Suchen...' and a 'Filter: do' button. Below the search is a folder icon labeled 'Physikversuche' with a '1' in a green circle, indicating one item. To the right is a microscope image with 'Bio' and 'Sci' tags, and a section titled 'Das Mikroskop' with the text 'In diesem Inhalt wird das Mikroskop dargestellt und deren Geschichte erklärt.'

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The screenshot shows a detailed view of a microscope. To the left is a large image of the microscope. To the right is a text block describing the history of microscopes. At the bottom, there are buttons for 'Inhalt bearbeiten' (Edit content) and 'Inhalt herunterladen' (Download content). A small smiley face icon is also present.

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The screenshot shows the 'Meine Inhalte' (My Content) section of the curricuLAB® PHYWE software. On the left, there's a sidebar with categories like 'Meine Inhalte' (2), 'Gemeinsame Inhalte' (145), 'PHYWE Bibliothek' (1644), 'MedienLB Bibliothek' (2500+), and 'Demo Inhalte' (11). In the center, there's a search bar and a folder icon with the number '1'. Below it is a section titled 'Physikversuche' (Physics Experiments) with a description: 'Physikversuche mit dem Thema Mechanik, Elektrizität und Magnetismus.' On the right, there's a context menu for a folder labeled 'Das Mikroskop' (The Microscope). The menu options include: Inhalt anzeigen (Show content), Inhalt bearbeiten (Edit content), Inhalt verschieben (Move content), Inhalt löschen (Delete content), Inhalt duplizieren (Duplicate content), Inhalt herunterladen (Download content), and Inhalt freigeben (Share content). There are also 'Bio' and 'Sci' icons at the bottom right.

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